

# A step by step guide

## How to test with the CoaguChek® INRange system

### Please review these tips for getting a good drop of blood.

Increasing the blood flow in your finger will help you get a good drop of blood:

- Warm your hand. Hold it under your arm, use a hand warmer, and/or wash with warm water.
- Let your arm hang by your side.
- Massage your finger from its base.




### 1. Wash Hands

Wash your hands in warm, soapy water. Make sure your fingertip is thoroughly dry.


### 2. Get Ready

Take a test strip out of the container. Close the container tightly. Note: You have 10 minutes to use a test strip once you remove it from the container.


### Set Targets

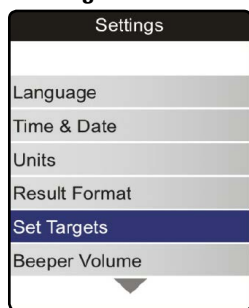


Select Set Targets and press the Enter button to open the corresponding menu.

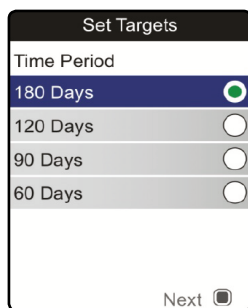


Select the desired time period for results to be displayed. Press the Enter button to confirm the selected time period.

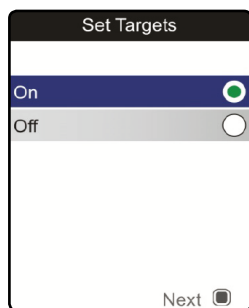

The current selected option is indicated by a green dot on the right.

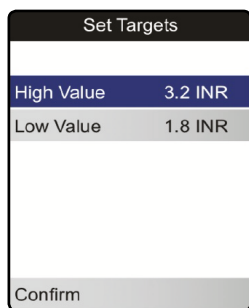

Select the value you wish to set and press the Enter button to proceed.

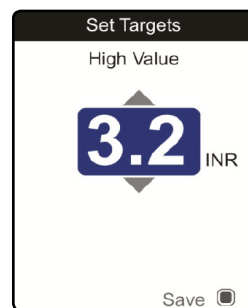


Repeat last steps to set the second value. Select Confirm and press the Enter button to save.

### Performing a test




After startup, Test is pre-selected. Press the enter button to start the test.

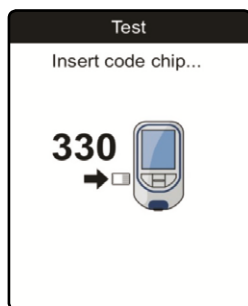


Insert code chip if prompted.




Slide the test strip into the guide in the direction of the arrows.

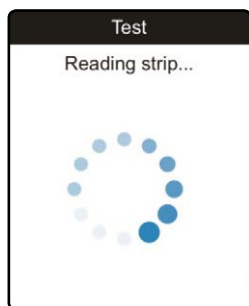

The meter reads the strip information (from the barcode on the test strip).

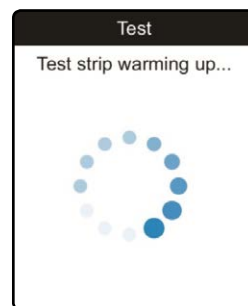

Once test strip is checked successfully, screen shows test strip warming up.

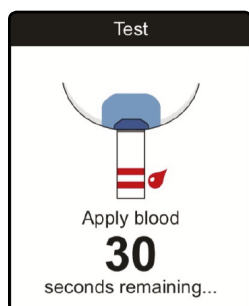


Once complete, you have 180 seconds to apply blood to the test strip.




Lance the side of a fingertip, apply the first blood drop to test strip.



Apply the blood to the sample application area within 15 seconds.

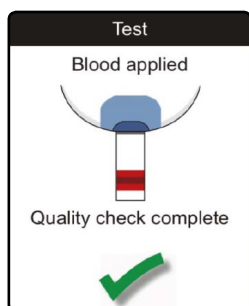

A beep tone indicates that you have applied enough blood.

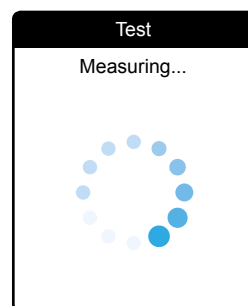

The meter now performs an automatic quality control check on the test strip.


# A step by step guide

## Interpreting and analysing your results


### Result Display




To add a comment, highlight **Add Comment** and go to the comment list.

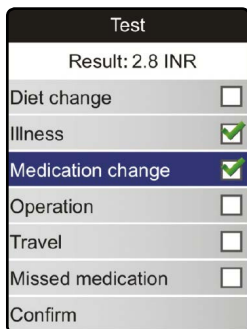

Use ▲▼ and ◻ to highlight, confirm and proceed with settings.

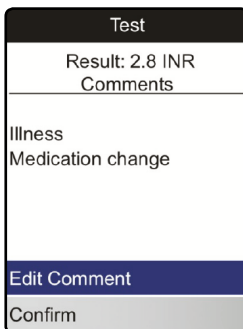

If the list of previously selected comments is correct, press ◻.

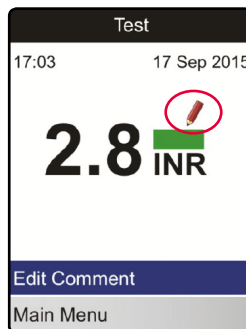


The result screen now shows the Comment symbol ✎. Test is complete.




Remove test strip, turn the meter off and dispose of the test strip.


### If your results are displayed in INR:

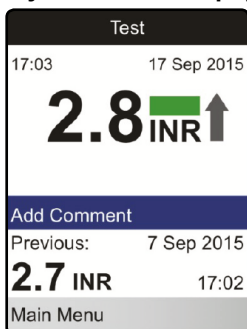

Green bar if within target range. (Arrow shows whether results are higher or lower than before.)

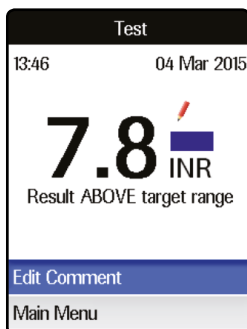


Blue bar if above target range (with words underneath).

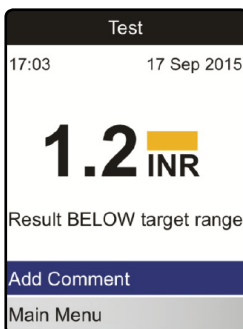

Yellow bar if below target range (with corresponding text).

### Displaying a Trend report

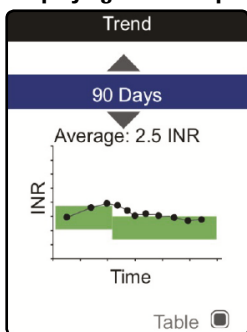

From the **Main Menu**, use ▲▼ to open the **My Results** menu, select **Trend** and press ◻ to display the report. The report above is shown as Graph view.


Date	Result
19 Jul 2015	2.5 INR
12 Jul 2015	2.6 INR
05 Jul 2015	2.8 INR
29 Jun 2015	2.4 INR
22 Jun 2015	2.9 INR
15 Jun 2015	2.9 INR

In Table view test results are shown in a list. The oldest date is shown at the bottom and the most recent date at the top. Use ▲▼ to select the time period you want.

Date	Result
19 Jul 2015	2.5 INR
12 Jul 2015	2.6 INR
05 Jul 2015	2.8 INR
29 Jun 2015	2.4 INR
22 Jun 2015	2.9 INR
15 Jun 2015	2.9 INR

Test results above or below the measuring range are marked accordingly. These test results appear in red in the table and are not displayed in the trend graph.

### Displaying a Target report

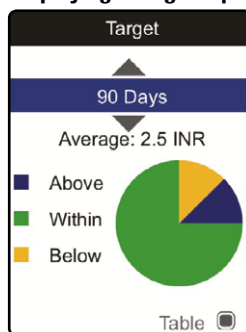

From the **Main Menu**, use ▲▼ to open the **My Results** menu, select **Target** and press ◻ to display the report.

Category	Percentage
Above	10%
Within	75%
Below	15%

Use ▲▼ to select the time period you want to evaluate. Press ◻ to toggle between graph and table display. Press the Back button to return to the **My Results** menu.

Freephone Care Line 0808 100 7666 (ROI 1 800 992 868)

The step by step guide is based on the CoaguChek INRange user manual (07408811001) V1. Please see full user manual for detailed instructions.

© 2021 Roche Diagnostics Limited. All rights reserved.  
COAGUCHEK and INRANGE are trademarks of Roche. All other trademarks or brand names are the property of their respective owners.

**Roche Diagnostics Limited**  
Charles Avenue, Burgess Hill, West Sussex, RH15 9RY. Company registration number: 571546.  
Date of preparation: February 2021. Document number: MC-IE-00930.

For use in the UK and Ireland only.

[www.roche.co.uk](http://www.roche.co.uk)

**CoaguChek®**